BIOSYSTEMS

# Leica HER2 FISH Control Slides

## Catalog No: TA9123

Leica Biosystems Newcastle Ltd  
Balliol Business Park West  
Benton Lane  
Newcastle Upon Tyne NE12 8EW  
United Kingdom  
☎ +44 191 215 4242

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### Intended Use

For *in vitro* diagnostic use.

Leica HER2 FISH Control Slides (TA9123) are designed for use with the Leica HER2 FISH System (TA9217) to validate assay performance. The Leica HER2 FISH Control Slides validate reagent performance only and do not verify test tissue preparation.

### Principle and Procedure

Each Leica HER2 FISH Control Slide contains four formalin-fixed, paraffin-embedded human breast cancer cell line cores that have been fully characterized for HER2 Gene amplification status, HER2 receptor load per cell and immunohistochemistry profile (as shown in Table 1).

| Cell Line  | Bond Oracle HER2 IHC System Profile | HER2 Receptor Load per Cell* | Leica HER2 FISH System HER2:CEP17 Acceptance Criteria |
|------------|-------------------------------------|------------------------------|---|
| SKBr-3     | 3+                                  | $4.3 \times 10^5$            | HER2 amplification is observed                        |
| MDA-MB-453 | 2+                                  | $1.4 \times 10^5$            | HER2:CEP17 gene ratio should be between 1.5 – 2.5     |
| MDA-MB-175 | 1+                                  | $6.3 \times 10^4$            | HER2 amplification is not observed                    |
| MDA-MB-231 | 0                                   | $9.3 \times 10^3$            | HER2 amplification is not observed                    |

\*HER2 receptor load analysis as assessed by flow cytometry.

Table 1: Leica HER2 FISH Control Slide Cell Line Profile

### Materials Provided

Five Leica HER2 FISH Control Slides are provided for use, each containing the four formalin-fixed, paraffin-embedded cell lines listed above.

### Materials Required But Not Provided

Refer to Leica HER2 FISH System (TA9217) Instructions for Use ([www.LeicaBiosystems.com/TA9217-IFU](http://www.LeicaBiosystems.com/TA9217-IFU)) for materials required but not provided.

### Recommendations for Use

It is recommended that a Leica HER2 FISH Control Slide is included in each staining run and with each new reagent lot to validate assay performance.

### Storage and Stability

The Leica HER2 FISH Control Slides are stable to their stated expiry date when stored at 2–8 °C. Do not use after stated expiration date.

### Quality Control

It is recommended that a Leica HER2 FISH Control Slide is included in each test run to monitor assay performance and to assess the accuracy of signal enumeration. A Leica HER2 FISH Control Slide should be run for each staining batch on the Leica BOND System and with each new reagent lot. In addition, individual users may choose to use their own control material.

Assess control slide quality and perform signal enumeration according to the instructions in the Signal Assessment and Enumeration section of the Leica HER2 FISH System (TA9217) Instructions for Use. The criteria for slide quality must be satisfied and the HER2:CEP17 ratio results should be within the established ranges for acceptable test performance.

### Patient Tissue

Patient specimens should be controlled according to standard laboratory operating procedures.

### Interpretation of Results

Refer to the Leica HER2 FISH System (TA9217) Instructions for Use and Interpretation Guide ([www.LeicaBiosystems.com/TA9217-IFU](http://www.LeicaBiosystems.com/TA9217-IFU)) for interpretation guidelines.

### Product Specific Limitations

Leica HER2 FISH Control Slides validate reagent performance only and do not verify test tissue preparation.

Minimal natural variation of gene amplification status will be seen within and between growth batches of cells. This variation is well within acceptable tolerance levels and does not affect the interpretation or performance of the system.

Leica HER2 FISH Control Slides have been prepared specifically for use with the Leica HER2 FISH System and have not been validated as a control for use with any other assay. They are **not** intended for use with other HER2 diagnostic devices such as the Leica Bond Oracle HER2 IHC System (TA9145).

## Warnings and Precautions

- For professional users only.
- As a rule, persons under 18 years of age are not allowed to work with this product. Users must be carefully instructed in the proper work procedure, the hazardous properties of the product and the necessary safety instructions.
- Specimens, before and after fixation, and all materials exposed to them, should be handled as if capable of transmitting infection and disposed of with proper precautions.
- Never pipette reagents by mouth and avoid contacting the skin and mucous membranes with reagents and specimens. If reagents or specimens come into contact with sensitive areas, wash with copious amounts of water. Seek medical advice. Consult federal, state or local regulations for disposal of any potentially toxic components.
- Minimize microbial contamination of reagents or an increase in non-specific staining may occur.
- Do not expose to temperature greater than 8 °C for extended periods of time.

## Troubleshooting

For technical assistance, contact your local Leica Biosystems' Technical Services Department or Distributor.

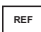
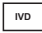
## Amendments to Previous Issue

Not applicable.

## Date of Issue

30 October 2014

## Symbol Identification

|  |   |   |                                   |   |                   |
|--|---|---|-----------------------------------|---|-------------------|
|  | Batch Code                                |  | Storage                           |  | Catalog number    |
|  | <i>In vitro</i> diagnostic medical device |  | Manufacturer                      |  | Fragile           |
|  | Consult instructions for use              |  | Contains sufficient for <n> tests |  | Use by YYYY-MM-DD |

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