



# Aperio GT 450 DX 1.1.1 and 1.1.2 Patch Notice

Leica Biosystems is always looking for ways to further improve our customers' experience using our products. Our software patches often include new features, but also include improvements based on customer input and our own internal testing. Please make sure to read below for the details of this patch so you can get the best out of your product. Thank you for being a customer of Leica Biosystems.

## Aperio GT 450 DX 1.1.1 Patch

### New Features Included in the 1.1.1 Patch

Case #	Component	Description
27657	Controller	Aztec barcode (English only) now supported.

### Updates Included in the 1.1.1 Patch

The following improvements were added in this patch.

Case #	Component	Description
27958	Console	The scanner operating system was updated to address latest security vulnerabilities.
27986	Console	An issue was corrected in which an invalid security warning message was generated.
27994	AutoLoader firmware	The AutoLoader slide handling error recovery process was updated to require user acknowledgement for scanner restart.
27999	Console	An issue was corrected in which barcode information could fail to be decoded by DataServer when adding the barcode to Aperio eSlide Manager if the barcode contained the & symbol.
28013	Console	A file formatting error resulting in duplicate slide entries within debug files used by Leica support for troubleshooting. This was corrected.
28018	Console	The scanner software now reports to the SAM event log if time is not being synchronized properly.

## Aperio GT 450 DX 1.1.2 Patch

This patch release updates Framegrabber drivers in the scanner.

## Aperio GT 450 DX 1.1.1 and 1.1.2 Patch Notice

Part Number/Revision: PN-0035, Revision A

Date: February 2023

This document applies to Aperio GT 450 DX Controller, Aperio GT 450 DX Console, and Aperio GT 450 DX SAM versions 1.1.1 and 1.1.2 and later


### Copyright Notice

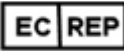
- ▶ Copyright © 2023 Leica Biosystems Imaging, Inc. All Rights Reserved. LEICA and the Leica logo are registered trademarks of Leica Microsystems IR GmbH. Aperio, GT, and GT 450 are trademarks of Leica Biosystems Imaging in the USA and optionally in other countries. Other logos, product and/or company names might be trademarks of their respective owners.
- ▶ This product is protected by registered patents. For a list of patents, contact Leica Biosystems.


### User Resources

- ▶ For the latest information on Leica Biosystems Aperio products and services, please visit [www.LeicaBiosystems.com/Aperio](http://www.LeicaBiosystems.com/Aperio).

### Contact Information – Leica Biosystems Imaging, Inc.

Headquarters	Customer Support
 Leica Biosystems Imaging, Inc. 1360 Park Center Drive Vista, CA 92081 USA Tel: +1 (866)-478-4111 (toll free) Direct International Tel: +1 (760) 539-1100	Contact your local support representative with any query and service request.  <a href="http://www.leicabiosystems.com/contact-us/">www.leicabiosystems.com/contact-us/</a>

European Union Authorized Representative	United Kingdom Responsible Person
 CEpartner4U Esdoornlaan 13 3951 DB Maarn The Netherlands	Leica Microsystems (UK) Limited Larch House, Woodlands Business Park Milton Keynes, England, United Kingdom, MK14 6FG

Importers	
 Leica Biosystems Deutschland GmbH Heidelberger Straße 17-19 69226 Nussloch, Germany	Leica Microsystems (UK) Limited Larch House, Woodlands Business Park Milton Keynes, England, United Kingdom, MK14 6FG

### Revision Record

Rev	Issued	Sections Affected	Detail
A	February 2023	All	New document.

